



Operating and Treatment Lights

NEXUS
PROXY
ONYX



NEXUS Operating light head

Modern Design


Operating lights INFIMED NEXUS, PROXY and treatment lights INFIMED ONYX with LED light source are distinguished by innovative technology and modern design.

The little weight, ergonomic light heads as well as comfortable handles enable **easy and precise** set up and positioning of the operating light during surgery.

Waterproof, resistant to environmental factors design and smooth surfaces guarantees **ease of disinfection** and maintenance for perfect hygiene.


Ergonomic shape of the light minimizes disruption of **laminar airflow** in the operating theatre. The light heads are closed construction, composed of solid, permanently assembled elements.

Flow-optimised light head

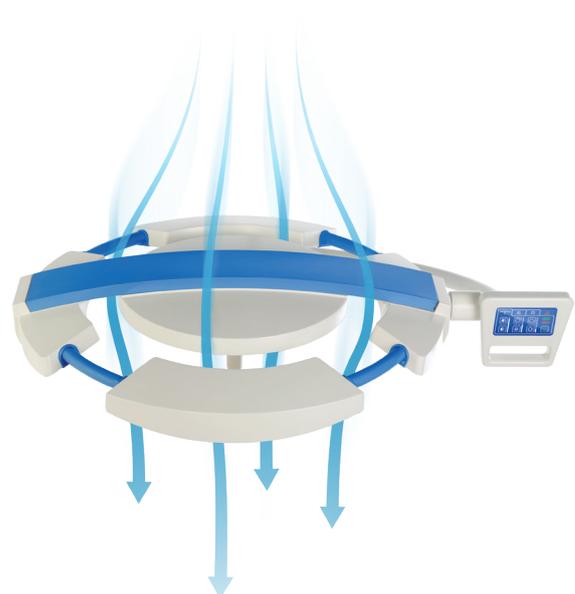


NEXUS, PROXY and ONYX


Operating and Treatment Lights

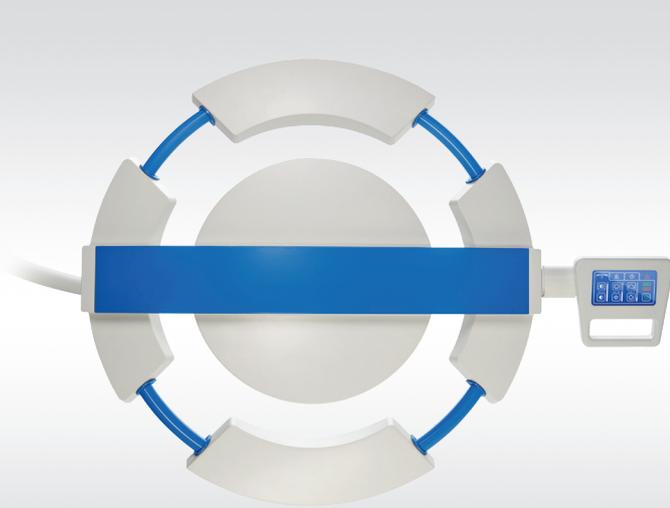

NEXUS Backside view

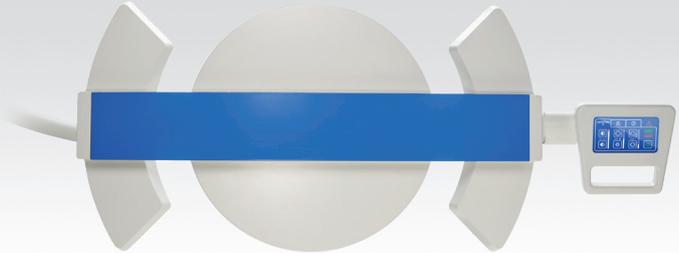


NEXUS Backside view

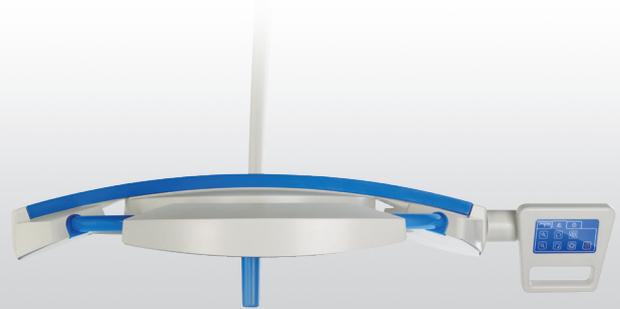

NEXUS Side view

Economy

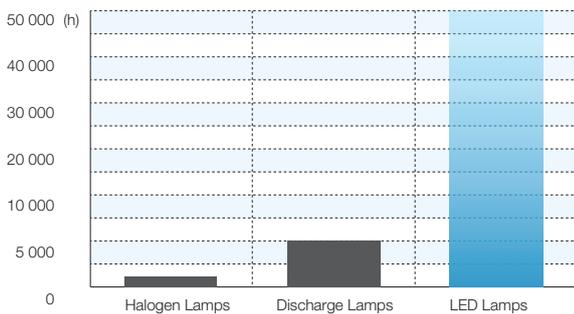

Low power consumption and high efficiency of LED light sources, provide measurable economic benefits by reducing maintenance costs of the operating theater.

Long lifetime of LED technology light sources, estimated at 60,000 hours, several times exceeds conventional light sources. There is no frequent replacement of the light sources, significantly reducing maintenance and replacement costs.

Environmental Protection



LED light sources do not contain any heavy metals like lead or mercury which facilitate their future utilization. Aluminium, the main material of LED's can be efficiently recycled. LED light sources does not produce harmful UV radiation.





NEXUS Operating light head

Efficiency

LED technology ensures **consistent light distribution** and enables almost shadow-free conditions.

Low heat radiation minimize the temperature increase in the surgeon's head area to imperceptible level.

High light intensity and color rendering index facilities surgeries of especially narrow and deep wounds. The illumination is electronically dimmed to provide optimum illumination of operating field. Together with wide range of **operating field size adjustment** optimal visual conditions are ensured.

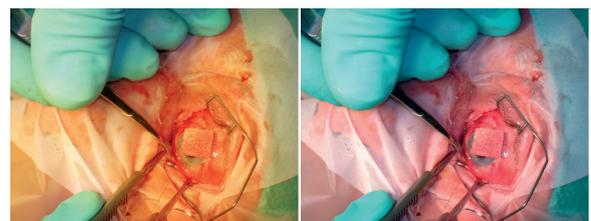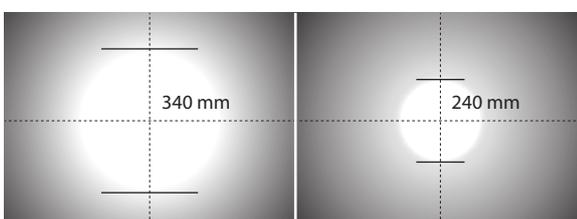

Colour temperature adjustment enables changing the contrast of the observed tissue and reduces fatigue of operating surgeon, which affects the comfort and ergonomics of the medical staff.



Adjustable **endo light** function reduces eye fatigue in case of endoscopic procedures and reflections on the monitors.

NEXUS

Operating light heads



NEXUS "Black" Operating light head

Parameter of operating light heads NEXUS	NEXUS OL-01	NEXUS OL-02	NEXUS OL-01 "BLACK"
Maximum illuminance $E_{c_{MI}}$ ($D_{MI} = 850$ mm)	160 000 lx	160 000 lx	160 000 lx
Central illuminance $E_{c_{Ref}}$ ($D_{Ref} = 1000$ mm)	130 000 lx	130 000 lx	140 000 lx
Light intensity adjustment	5 – 100%	5 – 100%	5 – 100%
Color temperature Tc (without adjustment)	4300K (4800 K)*	4300K (4800K)*	-----
Color temperature Tc (with adjustment)	3800-4800 K (3700 – 5000 K)*	-----	3700K – 5000K
Light field diameter d10 at Ec	240 – 340mm (200 - 360 mm)*	240 – 340mm (200 - 360 mm)*	160 – 360 mm
Working range	600 – 1500 mm	600 – 1500 mm	600 – 1500 mm
Illumination depth (L1+L2)	1300 mm	1300 mm	1300 mm
Color rendering index [Ra(1-8)]	> 95 (> 97)*	> 95 (> 97)*	> 97 (> 99)*
Red color rendering index [R9]	> 94	> 94	> 96
Endo lighting with adjustable intensity	Green (white)*, adjustable	Green (white)*, adjustable	Green (white)*, adjustable
Temperature of light surface after 60 min	< 40,00 ° C	< 40,00 ° C	< 40,00 ° C
Increase of temperature near doctor's head	< 1,00 ° C	< 1,00 ° C	< 1,00 ° C
Power supply	90 – 250 V AC	90 – 250 V AC	90 – 250 V AC
Power consumption	110W	80W	110W
Luminary head voltage	24 – 28V DC	24 – 28V DC	24 – 28V DC
Average service life of the LEDs (h)	> 60 000	> 60 000	> 60 000
Class of protection of light head	IP54	IP54	IP54

* additional options

Values are subject to a general industry tolerance of +/- 5%



PROXY Operating light head

Operating light PROXY

Is the smart solution of the modern operating light. The light head ensures perfect light parameters and full functionality, providing adjustments of light intensity, light field diameter and color temperature.

The light head can be equipped with the additional functions (as per page 10) as well as the video camera fixed in the center (as per page 11).

All configurations mentioned on pages 8-9 are possible to implement.



OL-03 / Operating light head with camera



PROXY Backside view

PROXY Side view

Parameter of operating light head PROXY	PROXY OL-03
Maximum illuminance $E_{c_{Mi}}$ ($D_{Mi} = 800 \text{ mm}$)	160 000 lx
Central illuminance $E_{c_{Ref}}$ ($D_{Ref} = 1000 \text{ mm}$)	120 000 lx
Light intensity adjustment	5 – 100%
Color temperature T_c (with adjustment)	3800 – 4800 K (3700-5000 K)*
Light field diameter d_{10} at E_c	240 – 340mm (200 - 360 mm)*
Working range	700 – 1400 mm
Illumination depth (L1+L2)	1200 mm
Color rendering index [Ra(1-8)]	>95 (>97)*
Red color rendering index [R9]	>93 (>95) *
Endo lighting with adjustable intensity	Green (white)*, adjustable
Temperature of light surface after 60 min	< 40,00 ° C
Increase of temperature near doctor's head	< 1,00 ° C
Power supply	90 – 250 V AC
Power consumption	80 W
Luminary head voltage	24 – 28V DC
Average service life of the LEDs (h)	> 60 000
Class of protection of light head	IP54





* Additional options

Values are subject to a general industry tolerance of +/- 5%


Configurations

Operating lights INFIMED are available in ceiling suspensions (single, double, triple), single wall suspension and the mobile version equipped with the battery.


Examples of suspension configurations for operating lights:

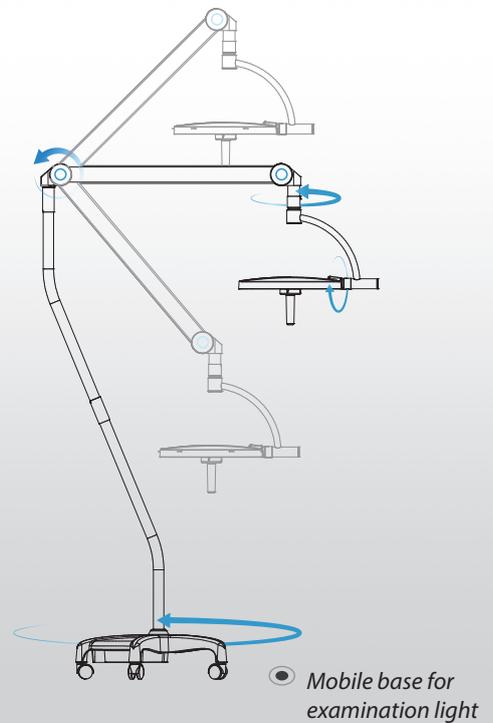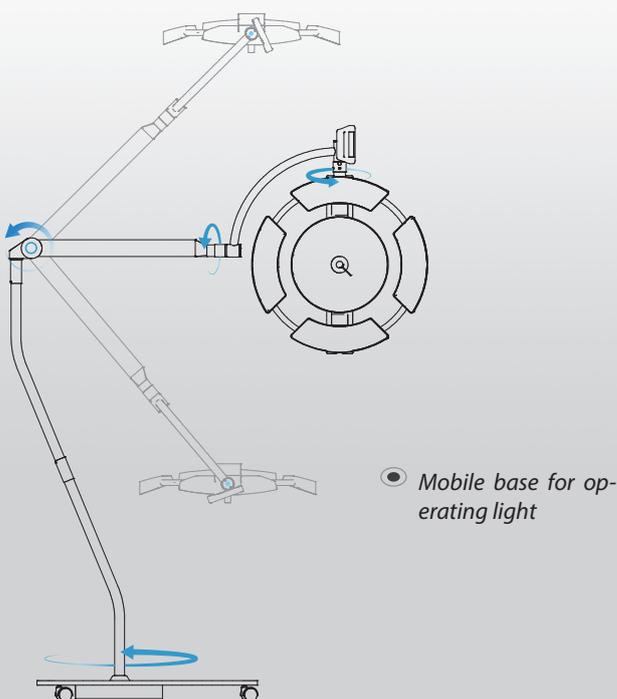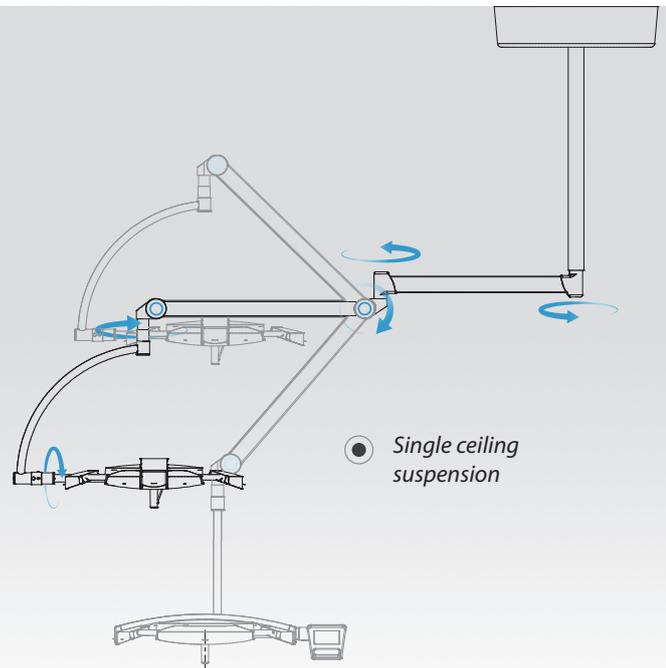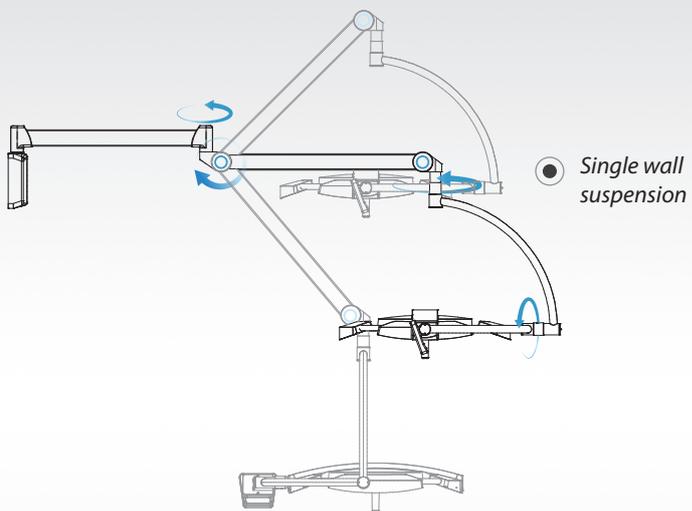


Configurations

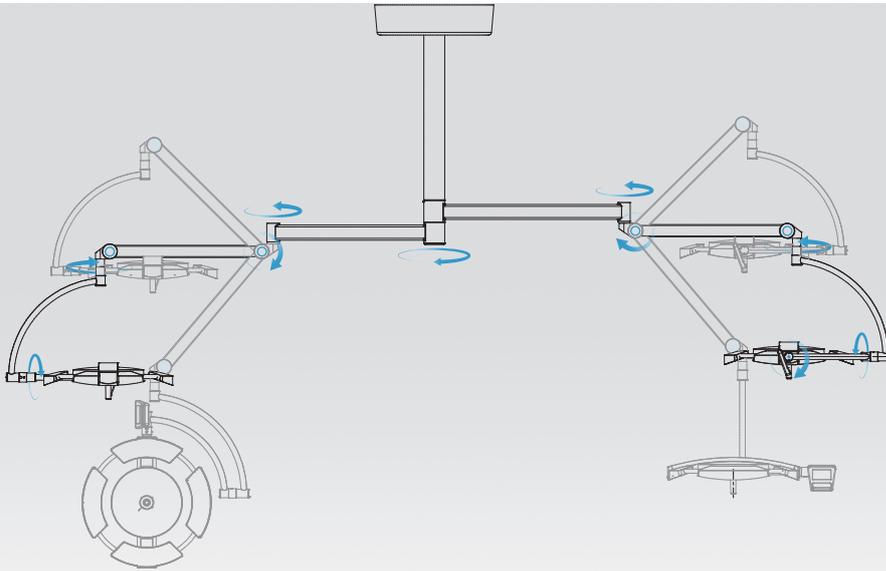


● Double ceiling suspension (>2700 mm) nx360°

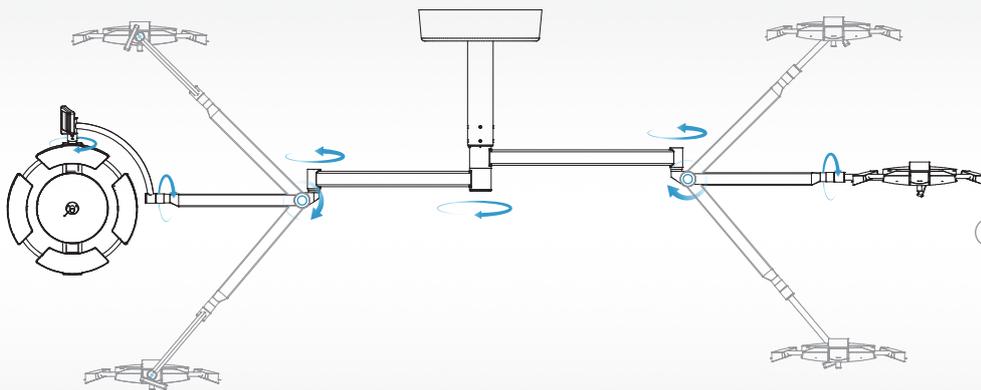

● Double ceiling suspension for low height rooms (<2700 mm) nx360°

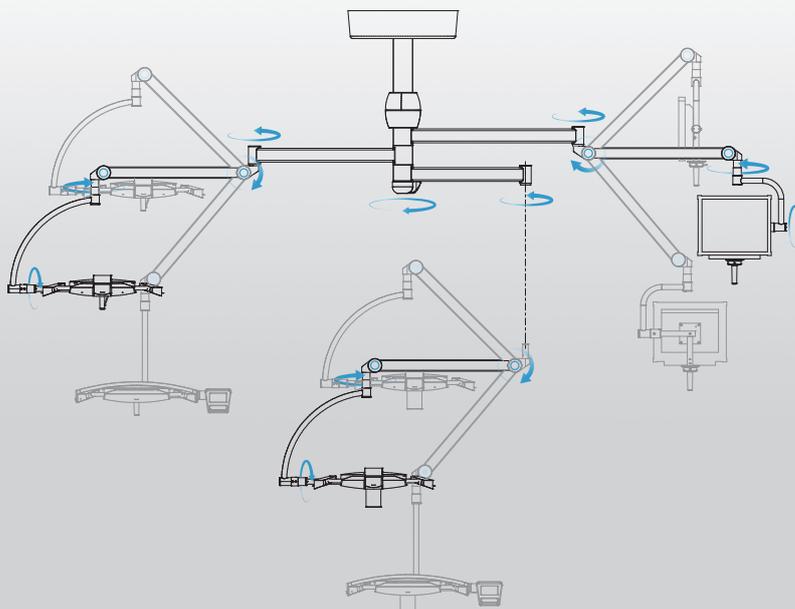

● Triple ceiling suspension, two arms n<360°, one arm nx360°

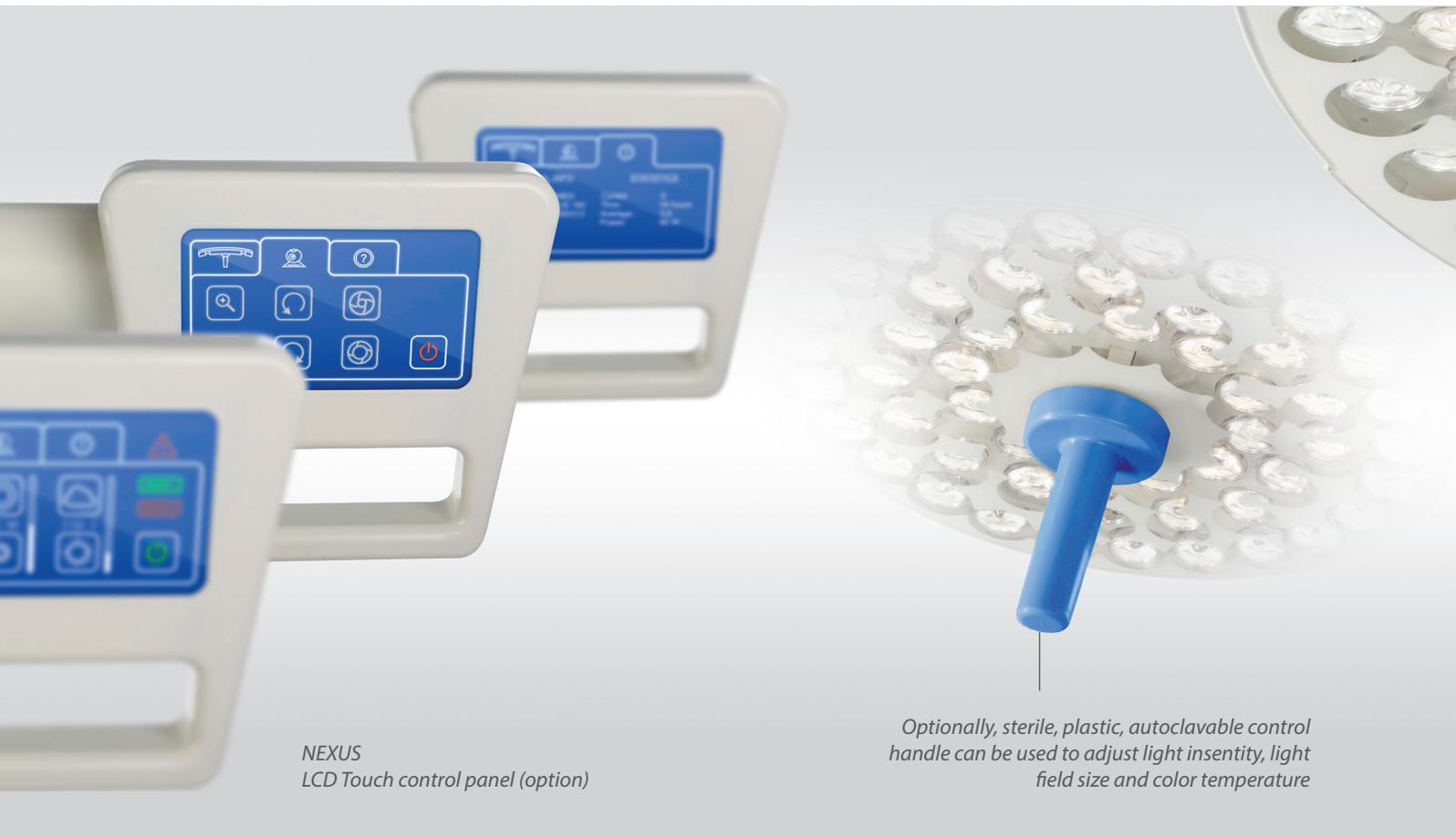

NEXUS
LCD Touch control panel (option)

Optionally, sterile, plastic, autoclavable control handle can be used to adjust light intensity, light field size and color temperature

Controls And Communication

The sensor type control panel (standard) enables adjustment of:

- illumination intensity
- endoscopic illumination intensity
- switching ON/OFF
- light field size (option)
- color temperature adjustment (option)



The LCD type control panel (option) additionally enables adjustment of camera functions (zoom, iris manual, auto, focus manual and auto, stop image, rotation with lock >360 deg.)

Additional functions

Measurement of the distance from operating area

The sensor in the light recognises the distance from the operating field and adjust the illumination intensity accordingly.

Intelligent shadow management

The sensor in the light perceives the surgeon head when it is directly under the light beam and brighten or dim specific LED's, in order to eliminate the shadows in the operating field.

Functions adjustment with sterile handle

Sterile, plastic, autoclavable control handle can be used to adjust 1-3 of following functions: light intensity, light field diameter and color temperature.

Synchronic adjustment of more light heads

Light heads adjustments can be synchronised: when adjusting any function of any light head, other light head (s) are adjusted simultaneously.

Infrared pointer

Infrared pointer shows the center of light beam.

Additional Equipment and Functions




Medical monitor



Integrated video system

Operating lights NEXUS and PROXY can be equipped with video camera, fixed in the center of the light. Optionally the lights can be delivered with standard sterile, plastic handle, to be exchanged into handle with camera in the future. Sony camera, Full 1920 x 1080/30p High Definition, adjustable from keyboard.

Camera Specifications	Details
Image Device	1/2.8 type Exmor™ CMOS sensor
Effective Pixels	Approx. 2 million
Digital Zoom	20x (200x with optical zoom)
Horizontal Viewing Angle	54.1° (wide end) to 2.9° (tele end)
Sync System	Internal
Electronic Shutter	1/2 to 1/10,000 s, 21 steps
White Balance	Auto
Focus System	Auto
Exposure Control	AE Control: Auto, Manual, Priority (shutter priority and iris priority)
Video Output	Details
Lens Value	20x Optical Zoom, f=4.7 mm (wide) ~ 94.0 mm (tele), f1.6 to f3.5
S/N Ratio More	More than 50 dB
Signal System	Signal System HD: 1080p/29.97, 1080p/25, 1080i/59.94, 1080i/50, 720p/50, 720p/29.97, 720p/25 SD: NTSC/PAL

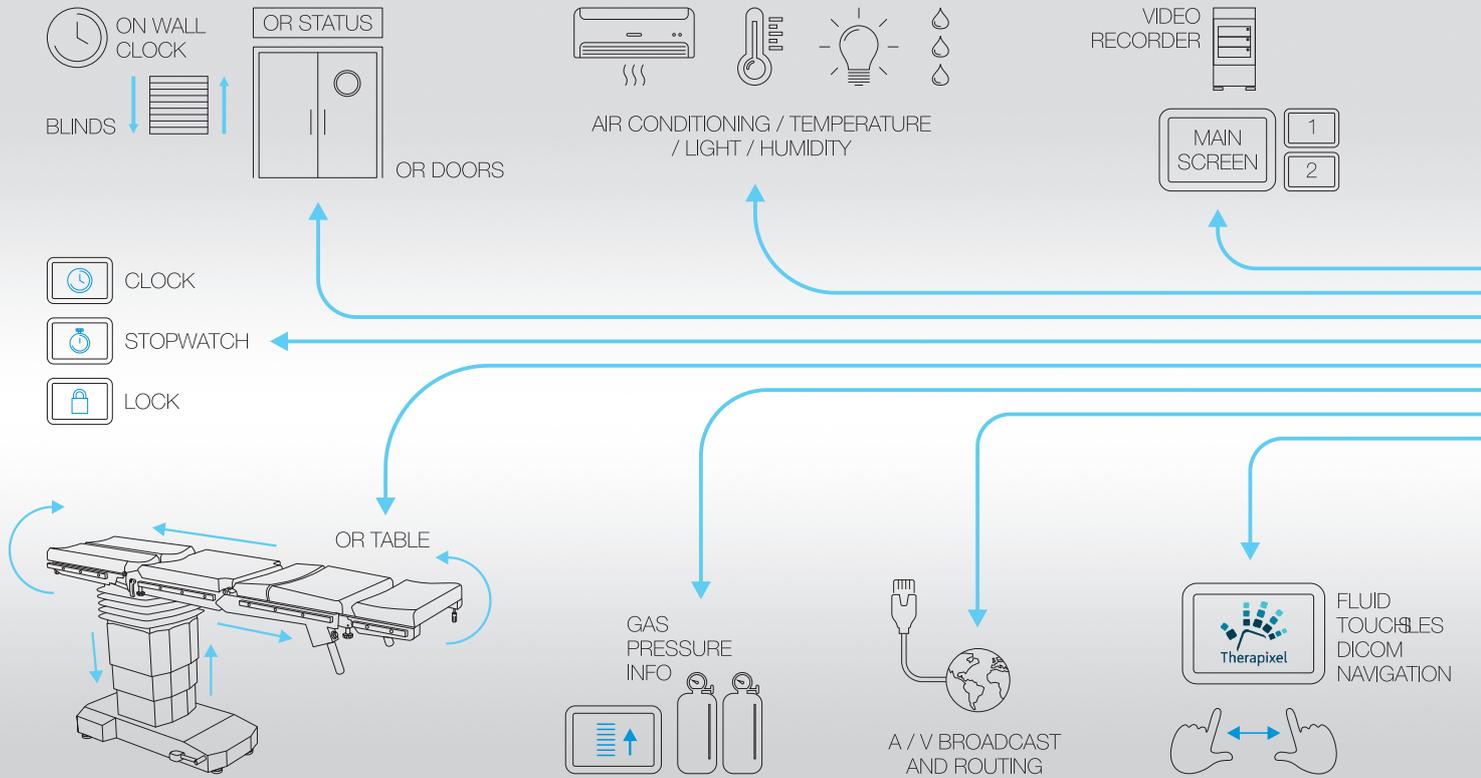







Opera control system

Provides better and easier control of the devices in the operating room from a single location. The system is very flexible, open, modular, can be adjusted to comply the client's expectations and requirements, accordingly to the above shown solutions. Can be operated from wall mounted control panel or wirelessly from mobile device.

The product is safe and secure and it is a certified Class I Medical Device manufactured in accordance with the ISO 9001, 13485 and other European standards. It provides a fully intuitive operation thanks to a simple and user-friendly interface, thereby minimizing the possibility of misuse and avoiding improper treatment operations.



Control System OPERA

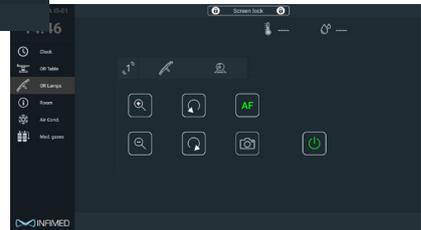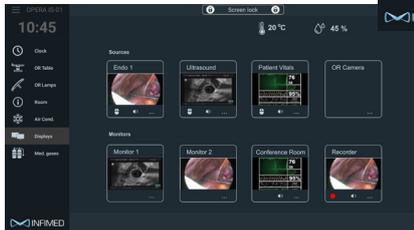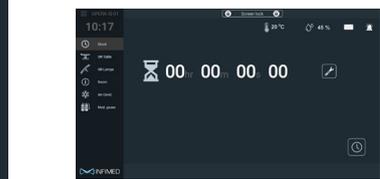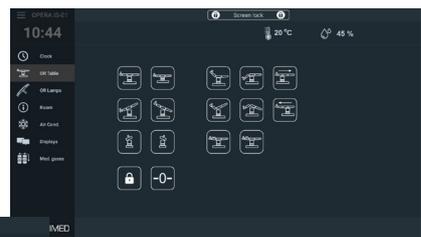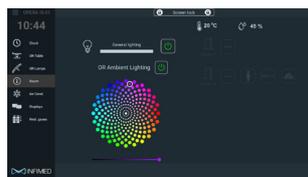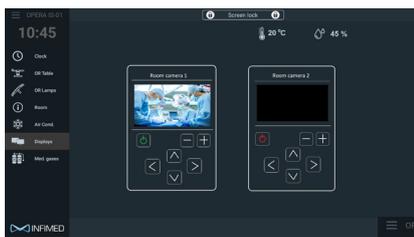



Treatment light ONYX, 60 000 lx

Lights ONYX

Lights ONYX are usually used in operating and procedures rooms, intensive care, neonatology units, induction rooms, dentist, dermatology, gynecology cabinets, veterinary cabinets, dissection rooms.

They are available with different levels of maximum illumination intensity: 30 000, 60 000, 100 000 up to 130 000 lx (for the lights having 100 000 up to 120 00 lx it is possible to add field diameter adjustment). Light head ONYX can be also used as minor operating light.

Light intensity and optional light field diameter adjustment are set up by control panel.

All configurations mentioned on pages 8-9 are possible to implement. Lights ONYX can be also used in double ceiling system as satellite with NEXUS or PROXY operating lights.



Control panel (standard)



Control panel (option)

ONYX

Minor operating and treatment light heads



Minor operating light ONYX, 100 000, 110 000, 120 000 lx with field size adjustment.



Treatment light ONYX, 30 000 lx

Parameter	ONYX TL-01 60 000/100 00-130 000 lx	ONYX TL-01 30 000 lx
Maximum illuminance $E_{c_{Mi}}$ ($D_{Mi} = 810$ mm)	90 000 lx (130 000 lx)	30 000 lx
Central illuminance $E_{c_{Ref}}$ ($D_{Ref} = 1000$ mm)	60 000 lx (100 000 lx)	60 000 lx
Dimming range	5 – 100%	5 – 100%
Color temperature Tc	4300K (4800K)*	4300K (4800K)*
Light field size d10 at Ec	240 mm / (240 – 340 mm)*	240 mm
Color rendering index [Ra(1-8)]	> 92 (> 95)*	>92 (>95)*
Red color rendering index [R9]	(>90) (> 92)*	(>90)*
Temperature of light surface after 60 min	< 40,00° C	< 40,00° C
Increase of temperature near doctor's head	< 1,00° C	< 1,00° C
Power supply	90 – 250 V AC	90 – 250 V AC
Power consumption ($\pm 10\%$)	50W	20W
Luminary heads voltage	24 – 28V DC	24 – 28V DC
Life cycle of the lights	> 50 000	> 50 000
Class of protection of light head	IP54	IPX2

* additional options
Values are subject to a general industry tolerance of +/- 5%

For special requirement of the customer it is possible to produce the product with changed technical parameters which do not diminish its safety.

The details given in this catalogue are correct at the time of going to press. Infimed Sp. z o.o., however, reserves the right to improve shown equipment.

adres INFIMED Sp. z o.o.
ul. Kabaty 1,
34-300 Żywiec,
Polska
tel/fax +48 33 861 40 96
email office@infimed.pl
www www.infimed.pl

NIP 553-251-29-67
REGON 243274947
KRS 0000463933

Sąd Rejonowy w Bielsku-Białej, VIII Wydział
Gospodarczy Krajowego Rejestru Sądowego.
Wysokość Kapitału Zakładowego: 500 000 PLN

bank Bank Spółdzielczy
w Węgierskiej Górze
SWIFT POLUPLPR

78 8131 0005 0016 2492 2000 0010 PLN
PL02 8131 0005 0016 2492 2000 0020 EUR
PL23 8131 0005 0016 2492 2000 0030 USD



 INFIMED

www.infimed.pl

